 IMMULITE<sup>®</sup>  
2000

## Phenobarbital

DPC<sup>®</sup>

# IMMULITE<sup>®</sup> 2000 Phenobarbital

---

## English

---

**Intended Use:** For *in vitro* diagnostic use with the IMMULITE 2000 Analyzer — for the quantitative measurement of phenobarbital in serum or heparinized plasma, as an aid in monitoring drug therapy.

Catalog Number: **L2KPB2** (200 tests),  
**L2KPB6** (600 tests)

Test Code: **PHE** Color: **Dark Green**

### Summary and Explanation

Phenobarbital is mainly used in the treatment of all seizures except absence seizures, in combination with other drugs such as carbamazepine, valproic acid and phenytoin.<sup>1,2</sup> The drug reduces synaptic transmission, thus, decreases excitability of the entire nerve cell although tolerance may develop with chronic usage.<sup>1,2</sup>

Phenobarbital is slowly but effectively absorbed after oral administration.<sup>1</sup> It reaches peak concentration after 4–10 hours. Half life is 70–100 hours in adults.<sup>1</sup> The optimal therapeutic range is 15–40 µg/mL, with toxic levels above 40 µg/mL.<sup>2</sup> Sedation is the most common adverse effect of the therapy.<sup>2</sup>

Phenobarbital is 40–60% bound to plasma protein.<sup>1</sup> It affects the clearance of other drugs by stimulating metabolism through induction of drug metabolizing enzymes, and/or by inhibiting metabolism through competition for a limited amount of enzymes.<sup>2</sup> Concentrations of carbamazepine, valproic acid are decreased by phenobarbital, whereas the effect of phenobarbital on the concentration of phenytoin is variable.<sup>2</sup> Valproic acid treatment increases phenobarbital concentration by decreasing phenobarbital's clearance.<sup>2</sup>

### Principle of the Procedure

IMMULITE 2000 Phenobarbital is a solid-phase, competitive chemiluminescent enzyme immunoassay.

**Incubation Cycles:** 1 × 30 minutes.

### Specimen Collection

The use of an ultracentrifuge is recommended to clear lipemic samples.

Hemolyzed samples may indicate mistreatment of a specimen before receipt by the laboratory; hence the results should be interpreted with caution.

Centrifuging serum samples before a complete clot forms may result in the presence of fibrin. To prevent erroneous results due to the presence of fibrin, ensure that complete clot formation has taken place prior to centrifugation of samples. Some samples, particularly those from patients receiving anticoagulant therapy, may require increased clotting time.

Blood collection tubes from different manufacturers may yield differing values, depending on materials and additives, including gel or physical barriers, clot activators and/or anticoagulants. IMMULITE 2000 Phenobarbital has not been tested with all possible variations of tube types. Consult the section on Alternate Sample Types for details on tubes that have been tested.

**Volume Required:** 20 µL serum or heparinized plasma.

**Storage:** 2 days at 2–8°C, or 1 month at –20°C.<sup>11</sup>

### Warnings and Precautions

For *in vitro* diagnostic use.

**Reagents:** Store at 2–8°C. Dispose of in accordance with applicable laws.

Follow universal precautions, and handle all components as if capable of transmitting infectious agents. Source materials derived from human blood were tested and found nonreactive for syphilis; for antibodies to HIV 1 and 2; for hepatitis B surface antigen; and for antibodies to hepatitis C.

Sodium azide, at concentrations less than 0.1 g/dL, has been added as a preservative. On disposal, flush with large volumes of water to prevent the buildup of potentially explosive metal azides in lead and copper plumbing.

**Chemiluminescent Substrate:** Avoid contamination and exposure to direct sunlight. (See insert.)

**Water:** Use distilled or deionized water.

## Materials Supplied

Components are a matched set. Labels on the inside box are needed for the assay.

### Phenobarbital Bead Pack (L2PB12)

With barcode. 200 beads, coated with monoclonal murine anti-phenobarbital antibody. Stable at 2–8°C until expiration date.

**L2KPB2:** 1 pack. **L2KPB6:** 3 packs.

### Phenobarbital Reagent Wedge (L2PBA2)

With barcode. 11.5 mL alkaline phosphatase (bovine calf intestine) conjugated to phenobarbital in buffer. Stable at 2–8°C until expiration date.

**L2KPB2:** 1 wedge. **L2KPB6:** 3 wedges.

Before use, tear off the top of the label at the perforations, without damaging the barcode. Remove the foil seal from the top of wedge; snap the sliding cover down into the ramps on the reagent lid.

### Phenobarbital Adjustors (LPBL, LPBH)

Two vials (Low and High), 2 mL each, of phenobarbital in human serum, with preservative. Stable at 2–8°C for 30 days after opening, or for 6 months (aliquotted) at –20°C.

**L2KPB2:** 1 set. **L2KPB6:** 2 sets.

Before making an adjustment, place the appropriate Aliquot Labels (supplied with the kit) on test tubes so that the barcodes can be read by the on-board reader.

## Kit Components Supplied Separately

**L2SUBM:** Chemiluminescent Substrate

**L2PWSM:** Probe Wash

**L2KPM:** Probe Cleaning Kit

**LRXT:** Reaction Tubes (disposable)

**SDCM:** Bi-level, human serum-based multi-constituent control.

Also required

Distilled or deionized water; test tubes; controls.

## Assay Procedure

Note that for optimal performance, it is important to perform all routine maintenance procedures as defined in the IMMULITE 2000 Operator's Manual.

See the IMMULITE 2000 Operator's Manual for: preparation, setup, dilutions, adjustment, assay and quality control procedures.

### Recommended Adjustment Interval:

2 weeks.

**Quality Control Samples:** Use controls or pools with at least two levels (low and high) of phenobarbital.

## Therapeutic Range

The following results are reported in the literature for the optimal therapeutic range and toxic levels of phenobarbital.<sup>1,9</sup>

	Phenobarbital µg/mL	Phenobarbital µmol/L
Nontoxic patients	15 – 40	65 – 174
Toxic patients	> 40	> 174

Consider these limits as *guidelines* only.

## Limitations

The following should be considered in the interpretation of the phenobarbital concentration:

- All concurrent medications taken by the patient
- The time of sampling in relation to the dosing history
- Existing diseases or conditions

Heterophilic antibodies in human serum can react with the immunoglobulins included in the assay components causing interference with *in vitro* immunoassays. [See Boscato LM, Stuart MC. Heterophilic antibodies: a problem for all immunoassays. Clin Chem 1988;34:27-33.] Samples from patients routinely exposed to animals or animal serum products can demonstrate this type of interference potentially causing an anomalous result. These reagents have been formulated to minimize the risk of interference; however, potential interactions between rare sera and test components can occur. For diagnostic

purposes, the results obtained from this assay should always be used in combination with the clinical examination, patient medical history, and other findings.

### Performance Data

See Tables and Graphs for data *representative* of the assay's performance. Results are expressed in µg/mL. (Unless otherwise noted, all were generated on serum samples collected in tubes without gel barriers or clot-promoting additives.)

#### Conversion Factor:

µg/mL × 4.34 → µmol/L

**Calibration Range:** 5 – 80 µg/mL  
(22 – 347 µmol/L).

**Analytical Sensitivity:** 1.0 µg/mL  
(4.3 µmol/L).

**Precision:** Samples were assayed in duplicate over the course of 20 days, two runs per day, for a total of 40 runs and 80 replicates. (See "Precision" table.)

**Linearity:** Samples were assayed under various dilutions. (See "Linearity" table for representative data.)

**Recovery:** Samples spiked 1 to 19 with three solutions (90, 165 and 330 µg/mL) were assayed. (See "Recovery" table for representative data.)

**Specificity:** Note that the crossreactivity with mephobarbital is 80%. (See "Specificity" table.)

**Bilirubin:** Presence of conjugated and unconjugated bilirubin in concentrations up to 200 mg/L has no effect on results, within the precision of the assay.

**Hemolysis:** Presence of hemoglobin in concentrations up to 512 mg/dL has no effect on results, within the precision of the assay.

**Lipemia:** Presence of triglycerides in concentrations up to 3,000 mg/dL has no effect on results, within the precision of the assay.

**Alternate Sample Type:** Samples ( $n = 15$ ) were collected into plain, heparinized and EDTA vacutainer tubes. All samples were spiked with phenobarbital and then assayed by the IMMULITE 2000 Phenobarbital procedure.

(Heparin) = 1.03 (Serum) – 1.29 µg/mL  
 $r = 0.988$

(EDTA) = 0.98 (Serum) – 0.34 µg/mL  
 $r = 0.995$

Means:

31.3 µg/mL (Serum)  
30.9 µg/mL (Heparin)  
30.3 µg/mL (EDTA)

**Method Comparison 1:** The assay was compared to DPC's IMMULITE Phenobarbital on 253 samples. (Concentration range: approximately 5 to 55 µg/mL. See graph 1.) By linear regression:

(IML 2000) = 1.07 (IML) – 0.47 µg/mL  
 $r = 0.995$

Means:

20.5 µg/mL (IMMULITE 2000)  
19.6 µg/mL (IMMULITE)

**Method Comparison 2:** The assay was also compared to Kit A on 135 samples. (Concentration range: approximately 5 to 65 µg/mL. See graph 2.) By linear regression:

(IML 2000) = 1.0 (Kit A) – 0.32 µg/mL  
 $r = 0.987$

Means:

23.1 µg/mL (IMMULITE 2000)  
23.4 µg/mL (Kit A)

### References

- 1) Moyer TP, Pippenger CE. Therapeutic Drug Monitoring. In: Burtis AC, Ashwood ER, eds. Tietz textbook of clinical chemistry, 2nd ed. Philadelphia: WB Saunders, 1994: 1094-154.
- 2) Drug Monitoring Data Pocket Guide II, 2nd ed. Washington, DC, AACC Press, 1994: 97-99.
- 3) Jolley ME. Fluorescence polarization immunoassay for determination of therapeutic drug levels in human plasma. *J Anal Tox* 1981;5:236-40.
- 4) Jolley ME, Stroupe SD, Schwenzler KS, et al. Fluorescence polarization immunoassay III. An automated system for therapeutic drug determination. *Clin Chem* 1981;27:1575-9.
- 5) Buchthal F, Lennox-Buchthal MA. Phenobarbital: Relation of serum concentration to control seizures. In: Woodbury DM, Penry JK, Schmidt RP, eds. Antiepileptic drugs. New York, NY: Raven Press, 1972: 335-43.
- 6) Glazko AJ. Antiepileptic drugs: Biotransformation, metabolism, and serum half-life. *Epilepsia* 1975;16:367-91.
- 7) Kutt H, Penry K. Usefulness of blood levels of antiepileptic drugs. *Arch Neurol* 1974;31:283-8.
- 8) Waddell WJ, Butler TC. The distribution and excretion of phenobarbital. *J Clin Invest* 1957;36:1217-26.
- 9) Burtis CA, Ashwood ER, editors. Tietz textbook of clinical chemistry. 3rd ed. Philadelphia: WB Saunders, 1995:874.
- 10) National Committee for Clinical Laboratory Standards. Procedures for the collection of diagnostic blood specimens by venipuncture; approved standard. 4th ed.

NCCLS Document H3-A4, Wayne, PA: NCCLS, 1998. 11) Quest Diagnostics. The 1997/1998 Quest Diagnostics Nichols Institute reference manual. San Juan Capistrano: Quest Diagnostics, 1997: 227.

### Technical Assistance

In the United States, contact DPC's Technical Services department.  
Tel: 800.372.1782 or 973.927.2828  
Fax: 973.927.4101. Outside the United States, contact your National Distributor.

The Quality System of Diagnostic Products Corporation is registered to ISO 13485:2003.

## Tables and Graphs

### Precision (µg/mL)

	Mean <sup>3</sup>	Within-Run <sup>1</sup>		Total <sup>2</sup>	
		SD <sup>4</sup>	CV <sup>5</sup>	SD	CV
1	5.0	0.15	3.0%	0.19	3.8%
2	13	0.29	2.2%	0.37	2.9%
3	19	0.39	2.1%	0.56	3.0%
4	39	0.66	1.7%	1.1	2.8%
5	80	2.7	3.4%	3.0	3.8%

### Specificity

Compound <sup>1</sup>	µg/mL Added <sup>2</sup>	% Cross reactivity <sup>3</sup>
Amobarbital	100	3.4%
Barbital	100	0.3%
Butabarbital	100	ND
5,5-Diallylbarbituric acid	100	0.7%
5-Ethyl-5(4-hydroxyphenyl) barbituric acid	40	13.5%
Hexobarbital	50	1.1%
Mephobarbital	40	80%
Pentobarbital	100	ND
Secobarbital	50	0.2%
Thiopental	250	0.2%

ND: Not detectable.<sup>4</sup>

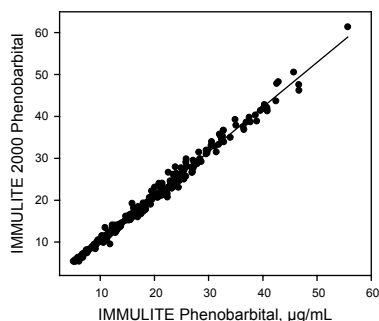
### Linearity (µg/mL)

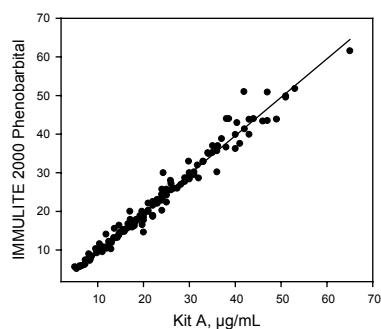
	Dilution <sup>1</sup>	Observed <sup>2</sup>	Expected <sup>3</sup>	%O/E <sup>4</sup>
1	8 in 8 <sup>5</sup>	39	—	—
	4 in 8	20	20	100%
	2 in 8	9.6	9.8	98%
	1 in 8	< 5.0	4.9	—
2	8 in 8	43	—	—
	4 in 8	21	22	96%
	2 in 8	10	11	91%
	1 in 8	6.0	5.4	111%
3	8 in 8	47	—	—
	4 in 8	23	24	96%
	2 in 8	13	12	108%
	1 in 8	6.6	5.9	112%
4	8 in 8	51	—	—
	4 in 8	25	26	96%
	2 in 8	13	13	100%
	1 in 8	6.8	6.4	106%
5	8 in 8	52	—	—
	4 in 8	23	26	89%
	2 in 8	12	13	92%
	1 in 8	6.7	6.5	103%
6	8 in 8	54	—	—
	4 in 8	27	27	100%
	2 in 8	15	14	107%
	1 in 8	6.0	6.8	88%

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O Sistema de Qualidade da Diagnostic Products Corporation está registado sob ISO 13485:2003.
EC REP DPC Biermann GmbH